


Hygosuc

EN

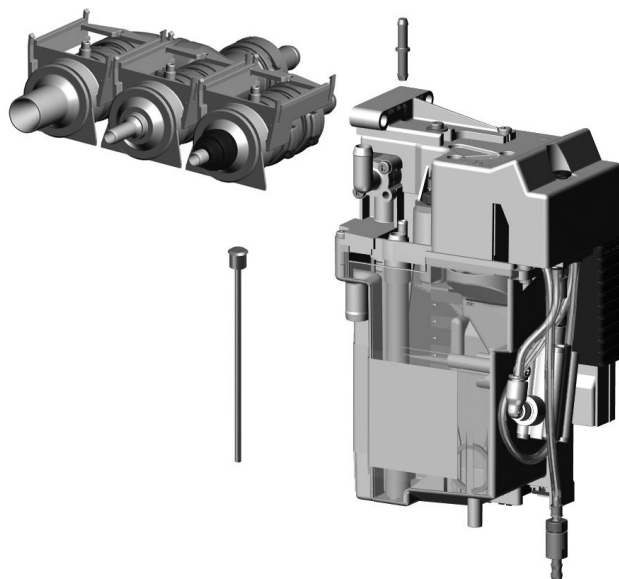

Installation and operating instructions

CE 0297

6005100079L02



 **DÜRR
DENTAL** 2111V005

Contents



Important information

1 About this document	3
1.1 Warnings and symbols	3
1.2 Copyright information	3
2 Safety	4
2.1 Intended purpose	4
2.2 Intended use	4
2.3 Improper use	4
2.4 General safety information	4
2.5 Specialist personnel	4
2.6 Electrical safety	4
2.7 Essential performance characteristics	5
2.8 Protection from threats from the Internet	5
2.9 Notification requirement of serious incidents	5
2.10 Only use original parts	5
2.11 Transport	5
2.12 Disposal	6



Product description

3 Overview	7
3.1 Scope of delivery	7
3.2 Accessories	7
3.3 Consumables	7
4 Technical data	8
4.1 Dosing module	8
4.2 CDS 1	11
4.3 CDS 60	12
4.4 Type plate	13
4.5 Evaluation of conformity	13
5 Operation	13
5.1 Dosing module	13
5.2 Suction points	14
5.3 CDS 1	14
5.4 CDS 60	14

5.5 Dosing and supply of a ready-to-use solution with disinfectant	14
5.6 Flush-cleaning with water	14
5.7 Continuous rinsing	15
5.8 Tyscor Pulse (optional)	15



Assembly

6 Requirements	16
6.1 Setup options	16
6.2 Connections	16
6.3 Hose materials	16
6.4 Installation requirements	16
7 Installation	17
7.1 Connection overview	17
7.2 Installation of the unit	19
7.3 Connecting the modules	21
7.4 Connecting the unit	21
8 Commissioning	22
8.1 Function test	22
8.2 Electrical safety checks	22
8.3 Combining devices safely	22
8.4 Monitoring the unit via the network	22



Usage

9 Operation	24
9.1 Dosing and supply of a ready-to-use solution with disinfectant	24
9.2 Flush-cleaning with water	25
9.3 Changing the disinfectant bottle	25
9.4 Button with indicator light	25
9.5 Cleaning the storage tank	26
9.6 Monitoring the unit via the network	26
9.7 Performing a visual check of the mixing ratio	27
10 Maintenance	28



Troubleshooting

11 Tips for operators 29

12 Tips for service technicians 30

13 Error messages in Tyscor Pulse 31

! Important information

1 About this document

These installation and operating instructions represent part of the unit.



If the instructions and information in these installation and operating instructions are not followed, Dürr Dental will not be able to offer any warranty or assume any liability for the safe operation and the safe functioning of the unit.

The German version of the installation and operating instructions is the original manual. All other languages are translation of the original manual. These installation and operating instructions apply to the Hygosuc CDS 1, order number: 6005-100-50 and Hygosuc CDS 60, order number: 6005100600.

1.1 Warnings and symbols

Warnings

The warnings in this document are intended to draw your attention to possible injury to persons or damage to machinery.

The following warning symbols are used:



General warning symbol

The warnings are structured as follows:



SIGNAL WORD

Description of the type and source of danger

Here you will find the possible consequences of ignoring the warning

- › Follow these measures to avoid the danger.

The signal word differentiates between four levels of danger:

- **DANGER**
Immediate danger of severe injury or death
- **WARNING**
Possible danger of severe injury or death
- **CAUTION**
Risk of minor injuries
- **NOTICE**
Risk of extensive material/property damage

Other symbols

These symbols are used in the document and on or in the unit:



Note, e.g. specific instructions regarding efficient and cost-effective use of the unit.



Refer to Operating Instructions.



CE labelling with the number of the notified body



Manufacturer



Air



Switch off and de-energise the device (e. g. unplug from mains).



Wear protective goggles.



Wear protective gloves.



Order number



Serial number



Dispose of correctly in accordance with EU Directive 2012/19/EU (WEEE).



Medical device



DC current



AC current

1.2 Copyright information

All circuits, processes, names, software programs and units mentioned in this document are protected by copyright.

The Installation and Operating Instructions must not be copied or reprinted, neither in full nor in part, without written authorisation from Dürr Dental.

2 Safety

Dürr Dental has designed and constructed this unit so that when used properly and for the intended purpose it does not pose any danger to people or property.

Despite this, the following residual risks can remain:

- Personal injury due to incorrect use/misuse
- Personal injury due to mechanical effects
- Personal injury due to electric shock
- Personal injury due to radiation
- Personal injury due to fire
- Personal injury due to thermal effects on skin
- Personal injury due to lack of hygiene, e.g. infection

2.1 Intended purpose

Dosing and supply of a ready-to-use disinfection solution for the subsequent cleaning, disinfection and care of a dental suction system.

2.2 Intended use

The product cannot operate by itself and must first be installed in a dental treatment unit before it can be used. It has been specially developed and tested for the use of Orotol Plus.

2.3 Improper use

Any other usage beyond this scope is deemed to be improper. The manufacturer accepts no liability for damage resulting from improper usage. In such cases, the user/operator will bear the sole risk.

The device must not be installed and operated in an environment with an enriched oxygen atmosphere.

2.4 General safety information

- › Always comply with the specifications of all guidelines, laws, and other rules and regulations applicable at the site of operation for the operation of this unit.
- › Check the function and condition of the unit prior to every use.
- › Do not convert or modify the unit.
- › Comply with the specifications of the Installation and Operating Instructions.
- › The Installation and Operating Instructions must be accessible to all operators of the unit at all times.

2.5 Specialist personnel

Operation

Unit operating personnel must ensure safe and correct handling based on their training and knowledge.

- › Instruct or have every user instructed in handling the unit.

Installation and repairs

- › Installation, readjustments, alterations, upgrades and repairs must be carried out by Dürr Dental or by qualified personnel specifically approved and authorized by Dürr Dental.

2.6 Electrical safety

- › Comply with all the relevant electrical safety regulations when working on the unit.
- › Never touch the patient and unshielded plug connections on the device at the same time.
- › Replace any damaged cables or plugs immediately.

Observe the EMC rules concerning medical devices

- › The unit is intended for use in professional healthcare facilities (in accordance with IEC 60601-1-2). If the appliance is operated in another environment, potential effects on electromagnetic compatibility must be taken into account.
- › Do not operate the unit in the vicinity of HF surgical instruments or MRT equipment.
- › Note that cable lengths and cable extensions have effects on electromagnetic compatibility.
- › No maintenance measures are required to maintain the EMV basic safety.
- › The emissions characteristics of this device render it suitable for use in industrial environments and hospitals (CISPR 11, Class A). When used in a residential environment (which normally requires Class B in accordance with CISPR 11), this device may not provide adequate protection from radio communication services. The operator may need to take corrective measures such as relocating or reorienting the device.

**NOTICE****Negative effects on the EMC due to non-authorised accessories**

- › Use only Dürr Dental parts or accessories specifically approved by Dürr Dental.
- › Using any other accessories may result in increased electromagnetic interference emissions or the unit having reduced electromagnetic immunity, leading to an erroneous operation mode.

**NOTICE****Erroneous operation mode due to use immediately adjacent to other devices or with other stacked devices**

- › Do not stack the unit together with other devices.
- › If this is unavoidable, the unit and other devices should be monitored in order to ensure that they are working correctly.

**NOTICE****Reduced performance characteristics due to insufficient distance between unit and portable HF communication devices**

- › Keep a distance of at least 30 cm between the unit (including parts and cables of the unit) and portable HF communication devices (wireless units) (including their accessories such as antenna cables and external antennas).

2.7 Essential performance characteristics

The Hygosuc device does not have any essential performance characteristics as set out in IEC 60601-1 (EN 60601-1) section 4.3.

The unit complies with the requirements according to IEC 60601-1-2:2014.

2.8 Protection from threats from the Internet

The unit is to be connected to a computer that can be connected to the Internet. Therefore, the

system needs to be protected from threats from the Internet.

- › Use antivirus software and update it regularly. Look for evidence of possible virus infection and, if applicable, check with the antivirus software and remove the virus.
- › Perform regular data backups.
- › Restrict access to units to trustworthy users, e.g. via a user name and password.
- › Make sure that only trustworthy content is downloaded. Only install software and firmware updates that have been authenticated by the manufacturer.

2.9 Notification requirement of serious incidents

The operator/patient is required to report any serious incident that occurs in connection with the device to the manufacturer and to the competent authority of the Member State in which the operator and/or patient is established/resident.

2.10 Only use original parts

- › Only use accessories and optional items that have been recommended or specifically approved by Dürr Dental.
- › Only use only original wear parts and replacement parts.

2.11 Transport

The original packaging provides optimum protection for the unit during transport.

If required, original packaging for the unit can be ordered from Dürr Dental.



Dürr Dental will not accept any responsibility or liability for damage occurring during transport due to the use of incorrect packaging, even where the unit is still under guarantee.

- › Only transport the unit in its original packaging.
- › Keep the packing materials out of the reach of children.



2.12 Disposal



An overview of the waste keys for Dürr Dental products can be found in the download area at:
www.duerdental.com
Document no.: P007100155

Unit



The unit may be contaminated. Instruct the company disposing of the waste to take the relevant safety precautions.

- › Decontaminate potentially contaminated parts before disposing of them.
- › Uncontaminated parts (e.g. electronics, plastic and metal parts etc.) should be disposed of in accordance with the local waste disposal regulations.
- › If you have any questions about the correct disposal of parts, please contact your dental trade supplier.



3 Overview

3.1 Scope of delivery

The following items are included in the scope of delivery (possible variations due to country-specific requirements and/or import regulations):

Hygosuc CDS 60 6005500102

- Dosiermodul
- Taste
- Saugstelle d=6 mm
- Saugstelle d=16 mm
- Saugstelle Schwallreinigung d=6/11/16 mm
- Schläuche
- Dürr Connect Teile

Hygosuc CDS 1 6005500100

- Dosiermodul
- Taste
- CDS 1 Pumpstation für Desinfektionsmittel
- CDS 1 Saugkopf
- Saugstelle d=6 mm
- Saugstelle d=16 mm
- Saugstelle Schwallreinigung d=6/11/16 mm
- Schläuche
- Dürr Connect Teile

3.2 Accessories

The following items are required for operation of the device, depending on the application:

Orotol plus 6x 1 litre bottle CDS110P5550

Orotol plus 30-l canister CDS110P9599

CDS 60 pumping station for disinfectant 0704-50

3.3 Consumables

The following materials are consumed during operation of the device and must be reordered separately:

Orotol plus 30-l canister CDS110P9599

Orotol plus 6x 1 litre bottle CDS110P5550

MD 555 cleaner special detergent for suction systems 6 x 1 l bottles/carton CCS555C5550

4 Technical data

4.1 Dosing module

Electrical data

Rated voltage	V AC/DC	24
Nominal current	A	0.13
Rated power	W	3
Frequency	Hz	50 / 60
Protection class		II
Type of protection		IP 20

Communication interfaces

Number of switched outputs		2
Max. switching current	mA	400

General technical data

Dimensions (W x H x D)	mm	105 x 168 x 65
Max. filling capacity	ml	250
Weight	kg	0.36
Compressed air pressure	bar (hPa)	3 - 5 (3000 - 5000)
Water pressure	bar (hPa)	3 - 5 (3000 - 5000)
Water hardness	°dH	up to 12
Water flow rate of the water supply*	l/min	>0.5
Water flow rate in the device at a water pressure of 5 bar*	l/min	approx. 0.42
Water flow rate with a free outlet at 3 bar*	l/min	>0.5
Disinfectant pressure	bar (hPa)	0.5 - 4 (500 - 4000)

*Flow rates are provided as guide values and may vary depending on the installation situation.

Ambient conditions during storage and transport

Temperature	°C	-30 to +60
Relative humidity	%	< 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Relative humidity	%	< 70

Classification

Medical Device Class		Ila
----------------------	--	-----



Electromagnetic compatibility (EMC) Interference emission measurements

High-frequency emissions in accordance with CISPR 11	Group 1 Class A
Interference voltage at the power supply connection CISPR 11:2009+A1:2010	Compliant
Electromagnetic interference radiation CISPR 11:2009+A1:2010	Compliant

Electromagnetic compatibility (EMC) Interference immunity measurements cover

Immunity to interference, discharge of static electricity IEC 61000-4-2:2008 ± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Compliant
Immunity to interference, high-frequency electromagnetic fields IEC 61000-4-3:2006+A1:2007+A2:2010 3 V/m 80 MHz - 2.7 GHz 80% AM at 1 kHz	Compliant
Immunity to interference, near fields of wireless HF communication devices IEC 61000-4-3:2006+A1:2007+A2:2010 See immunity to interference table, near fields of wireless HF communication devices	Compliant
Immunity to power frequency magnetic fields IEC 61000-4-8:2009 30 A/m 30 Hz or 60 Hz	Compliant

Immunity to interference table, near fields of wireless HF communication devices

Radio service	Frequency band MHz	Test level V/m
TETRA 400	380 - 390	27
GMRS 460 FRS 460	430 - 470	28
LTE band 13, 17	704 - 787	9
GSM 800/900 TETRA 800 iDEN 820 CDMA 850 LTE band 5	800 - 960	28

Immunity to interference table, near fields of wireless HF communication devices

Radio service	Frequency band MHz	Test level V/m
GSM 1800 CDMA 1900 GSM 1900 DECT LTE band 1, 3, 4, 25 UMTS	1700 - 1990	28
Bluetooth WLAN 802.11 b/g/n RFID 2450 LTE band 7	2400 - 2570	28
WLAN 802.11 a/n	5100 - 5800	9

Electromagnetic compatibility (EMC)
Interference immunity measurements supply input

Immunity to interference, rapid transient bursts – AC voltage grid

IEC 61000-4-4:2012

± 2 kV

100 kHz repetition frequency

Compliant

Immunity to interference by surges, line-line

IEC 61000-4-5:2005

± 0.5 kV, ± 1 kV

Compliant

Immunity to interference by surges, line-earth

IEC 61000-4-5:2005

± 0.5 kV, ± 1 kV, ± 2 kV

Compliant

Immunity to interference, line-conducted disturbances induced by high-frequency fields – AC voltage grid

IEC 61000-4-6:2013

3 V

0.15 - 80 MHz

6 V

ISM frequency bands

0.15 - 80 MHz

80% AM at 1 kHz

Compliant

Electromagnetic compatibility (EMC)
Interference immunity measurements SIP/SOP

Immunity to interference, discharge of static electricity

IEC 61000-4-2:2008

± 8 kV contact

± 2kV, ± 4 kV, ± 8 kV, ± 15 kV air

Compliant

Immunity to interference, rapid transient bursts – I/O,

SIP/SOP ports

IEC 61000-4-4:2012

± 1 kV

100 kHz repetition frequency

Compliant



Electromagnetic compatibility (EMC) Interference immunity measurements SIP/SOP

Immunity to interference, line-conducted disturbances
induced by high-frequency fields – SIP/SOP ports
IEC 61000-4-6:2013

3 V

0.15 - 80 MHz

Compliant

6 V

ISM frequency bands

0.15 - 80 MHz

80 % AM at 1 kHz

4.2 CDS 1

Electrical data

Rated voltage	V AC/DC	24
Nominal current	A	0.25
Rated power	W	6
Protection class		II
Type of protection		IP 20
Mains frequency	Hz	50 / 60

Communication interfaces

Number of inputs	1
------------------	---

General technical data

Dimensions (W x H x D) Pump module	mm	55 x 88 x 65
Dimensions (W x H x D) Suction head with disinfectant bottle	mm	90 x 230 x 76
Weight	kg	0.2
Suction height of the pump	m	-1
Delivery height of the pump	m	5
Delivery	ml/min	100 - 340
Operating pressure	MPa	0.05 - 0.1

Ambient conditions during storage and transport

Temperature	°C	-30 to +60
Relative humidity	%	< 95

Ambient conditions during operation

Temperature	°C	+10 to +40
Relative humidity	%	< 70



4.3 CDS 60

Electrical data

Voltage	V	24
Frequency	Hz	50 / 60
Nominal current	A	0.13
Rated power	W	3
Protection class		II

Membrane pump

Voltage	V DC	24
Output	W	75
Rated operating pressure	bar (MPa)	2 - 4 (0.2 - 0.4)
Factory setting	bar (MPa)	3 (0.3)
Operating mode		S3 (2 h ON, 2 h OFF)
Treatment stations (SF 60%)	max.	50 (30)
SF = simultaneity factor		


General data

Dimensions (H x W x D)	cm	105 x 64 x 61
Weight with pallet incl. packaging, approx.	kg	140
Noise level	dB(A)	60 ±1.5

4.4 Type plate


Basic unit

The type plate is located on the rear of the device.

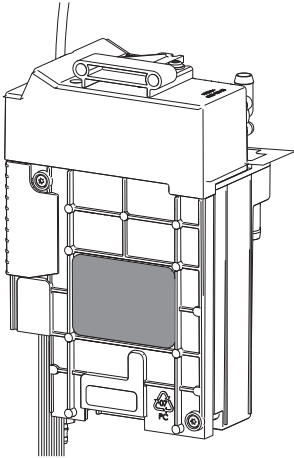

CDS 1

The type of the CDS 1 is located on the pump module.

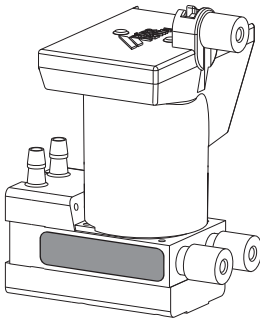

4.5 Evaluation of conformity

This device has been subjected to conformity acceptance testing in accordance with the current relevant European Union guidelines. This equipment conforms to all relevant requirements.


5 Operation

The device consists of two modules:

- Dosing module
- Suction points

In addition, a pump station for disinfectant (CDS 1 or CDS 60) is also required.

5.1 Dosing module

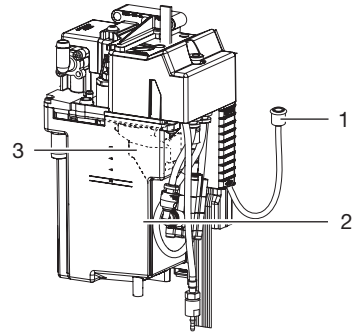

- 1 Button
- 2 Storage tank (250 ml)
- 3 Dosing tank (4 ml)

After the unit is switched on, the storage tank is filled with 250 ml water for continuous rinsing and flush-cleaning.

During the dosing and supply of a ready-to-use solution, the dosing module mixes the disinfectant with water and provides 5x 200 ml disinfectant solution.

Display/handling

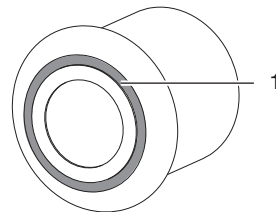


Fig. 1: Button with indicator light

- 1 Indicator light

The button is used for the following functions:


- Operating the unit
 - Starting dosing and supply of a ready-to-use solution
 - Acknowledging errors
- Indicating the various unit statuses

The button indicates the following statuses:

Indicator light	Status
Lights up green	Unit ready
Flashing rhythmically – green	Unit running
Flashing non-rhythmically – green	Reaction time running
Flashing red	No disinfectant
Lights up red	Fault

Indicator light statuses (see "9.4 Button with indicator light").


5.2 Suction points

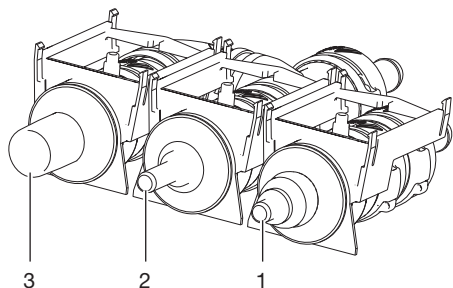

- 1 Flush-cleaning connection for all suction hoses
- 2 Small suction hose connection for cleaning and disinfection
- 3 Large suction hose connection for cleaning and disinfection

5.3 CDS 1

The pumping station for disinfectant consists of a pump module and a suction head.

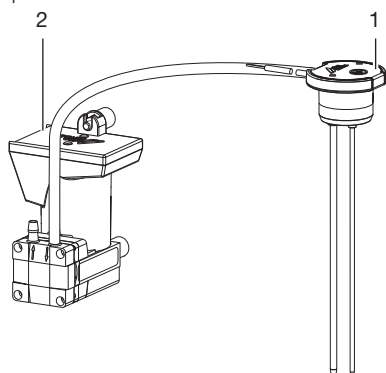

- 1 Suction head
- 2 Pump module

The suction head is connected to the disinfectant bottle.

The pump module transports the disinfection concentrate to the dosing module.

5.4 CDS 60

The central disinfection station provides disinfection concentrate for the cleaning and disinfection of dental treatment units. The disinfection concentrate must be mixed with water in the treatment unit.

5.5 Dosing and supply of a ready-to-use solution with disinfectant

Once all suction hoses are connected to the suction points, press the button to start the dosing and supply of the ready-to-use solution – the button will then flash green (rhythmically).

1. Rinsing with water:
The suction hoses are rinsed 5 times with 200 ml water.
2. Rinsing with disinfectant solution:
In the dosing module, 5x 200 ml disinfectant solutions (each made up of 4 ml disinfectant and 196 ml water) are mixed and aspirated through the suction hoses.
3. Reaction:
During the reaction time (one hour) the button flashes green (non-rhythmically).
If the treatment unit is switched off during the reaction time, the duration and end of the reaction time are not recorded and documented.

5.6 Flush-cleaning with water

Perform flush-cleaning on the suction hose used after each treatment.

- Hold the suction hose on the blue flush-cleaning connection and hold it firmly in place here during the flush-cleaning.
Water is aspirated from the storage tank. If the storage tank is empty, the button flashes green (rhythmically).
- Detach the suction hose from the flush-cleaning connection.

Result:

The storage tank refills with water. The button lights up green (ready for operation).

5.7 Continuous rinsing

If the suction hoses of the treatment unit are in use, the suction system is continuously supplied with water from the storage tank via the suction unit vacuum.


This prevents drying out and therefore the build-up of deposits in the suction system that would be difficult to dissolve.

5.8 Tyscor Pulse (optional)

The software is installed on a computer in the local practice network and is connected to the devices of Dürr Dental in the practice.

If there is a message for a device in the practice, the software transfers the message to the cloud (vistasoftmonitor.com). In addition, a message of the VistaSoft Monitor Notifier is shown in the taskbar.


The current status of the devices as well as messages and errors can be viewed using a browser.

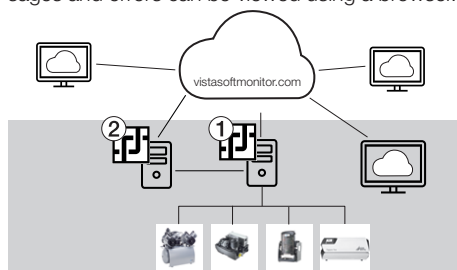

- 1 PC in local network with server installation
- 2 PC in local network with client installation (optional)



The **cockpit** shows the devices with the current characteristic data and provides a quick overview of the functional status of the devices.

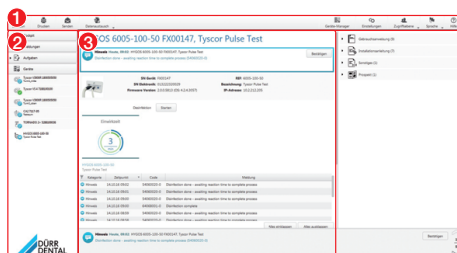The software interface consists of the menu bar, the side bar and the contents area.



- 1 Menu bar
- 2 Side bar
- 3 Contents area

The contents area depends on the tab selected on the side bar. The current messages are always displayed in the lower part of the contents area.

If there are several current messages, then the mouse wheel or the  or  buttons can be used to scroll through the messages.



The views and rights depend on the selected access level (Operator, Administrator or Service Technician).

While the software is running (even if the software window is closed), the access level is visible in the task bar (or Mac OS menu bar). The symbol shows the current status of the devices (see "Querying messages"). If a new message appears, a speech bubble tip also appears.

6 Requirements

6.1 Setup options

- Installation in treatment units in dental surgeries or dental clinics.

6.2 Connections



The device has no main power switch. Power is supplied to the device from the dental treatment unit. The device is disconnected from the power supply via the main power switch of the dental treatment unit or via the main power switch of the practice.

The connections comply with the specified values (see "4 Technical data"):

- Electrical connection 24 V AC/DC / 0.13 A / 3 W / IP 20
- The supply voltage to the device must satisfy the requirements for two means of patient protection (MOPP) as set out in IEC 60601-1 in relation to the supply network.
- Compressed air connection 3 – 5 bar
- Water connection 3 – 5 bar / up to 12°dH
- Disinfectant connection
CDS 60: 0.5 - 4 bar

6.3 Hose materials

For waste connections and suction lines only use the following hose types:

- Flexible spiral hoses made of PVC with integrated spiral or equivalent hoses
- Hoses that are resistant to dental disinfectants and chemicals




Plastic hoses will display signs of ageing over time. Therefore, they should be inspected regularly and replaced as necessary.

The following types of hoses must not be used:

- Rubber hoses
- Hoses that are not sufficiently flexible

6.4 Installation requirements

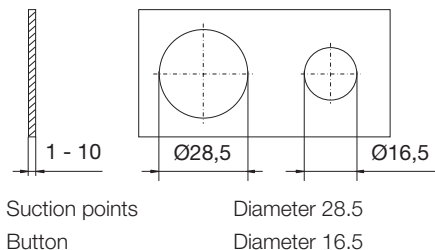

7 Installation

7.1 Connection overview

Dosing module



NOTICE

Equipment damage due to use of unauthorised hose material

➤ Only use hose material specifically approved by Dürr Dental.

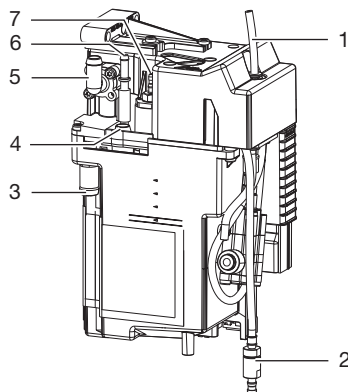


Fig. 2: Dosing module connections

1	Compressed air connection	mm	2 x 1 x 4
2	Disinfectant connection	mm	4 x 2 x 8
3	Overflow connection, internal diameter 10 mm	mm	10 x 1.8 x 13.6
4	Suction point connections for flush-cleaning	mm	7 x 2 x 11
5	Suction point connection for aspiration of the ready-to-use disinfectant solution	mm	7 x 2 x 11
6	Continuous rinsing connection	mm	3 x 1.5 x 6
7	Water connection (blue), max. length 6 m	mm	2 x 1 x 4

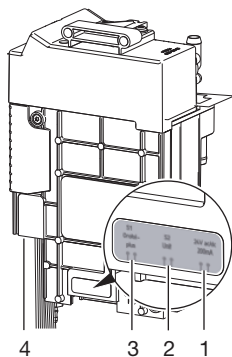


Fig. 3: Dosing module connections (rear)

- 1 Supply 24 V AC/DC
- 2 Controlled output e.g. for displaying the reaction time
- 3 Controlled output for disinfectant supply (e.g. CDS 1 or CDS 60)
- 4 Network connection

CDS 1

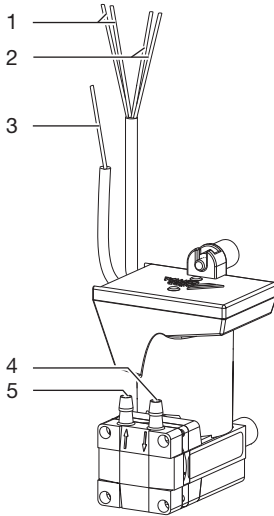


Fig. 4: Connections, CDS 1 pump module

- 1 Inputs for switch signal (brown and orange)
- 2 Supply 24 V AC/DC (red and black)
- 3 Cable, CDS 1 pump module to suction head
- 4 Intake connector (to the suction head)
- 5 Outlet connector (to the dosing module)

7.2 Installation of the unit



NOTICE


Danger to components due to electrostatic discharge (ESD)

- › Establish equipotential bonding between the person carrying out the work and the surroundings, e.g. via a wrist strap that is connected to the basic unit. Here, the basic unit must be earthed via the earth terminal.

Dosing module

- › Drill two holes at a suitable position in the treatment unit.


- › Attach the dosing module using two screws.

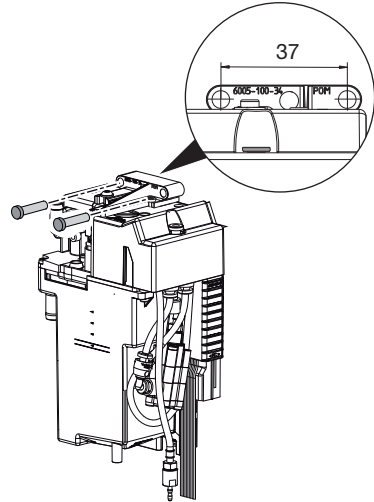

When doing so, make sure that the dosing module is positioned lower (up to 25 cm) than the hose manifold.


The holder can be rotated by 90° for attachment purposes.

- › Loosen the screws, remove the holder and rotate by 90°.

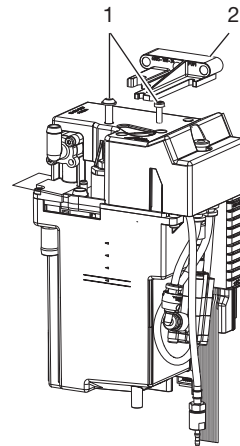

- 1 Screws
- 2 Owner

- › Fasten the screws.

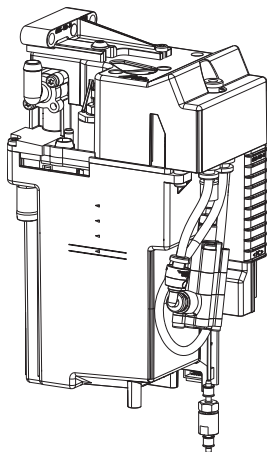

CDS 1

Observe the position of the CDS 1 modules and Hygosuc dosing module:

- Top edge of the suction head (disinfectant bottle) underneath the dosing module (distance < 1 m)
- Distance between pump module and dosing module < 5 m

Mounting the disinfectant bottle with suction head


The following must be taken into account when positioning the disinfectant bottle:

- Horizontal base
- Secured against tipping over, e.g. using a retaining bracket
- Ease of accessibility for changing

CDS 1 pump module

- › Drill three holes with a diameter of 4.5 mm in a metal plate.

- › Attach the pump with M4 screws to the vibration dampers.

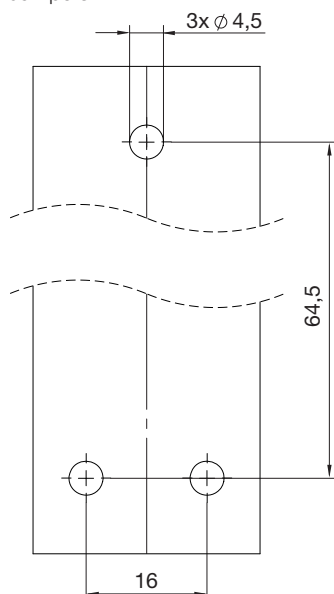

7.3 Connecting the modules

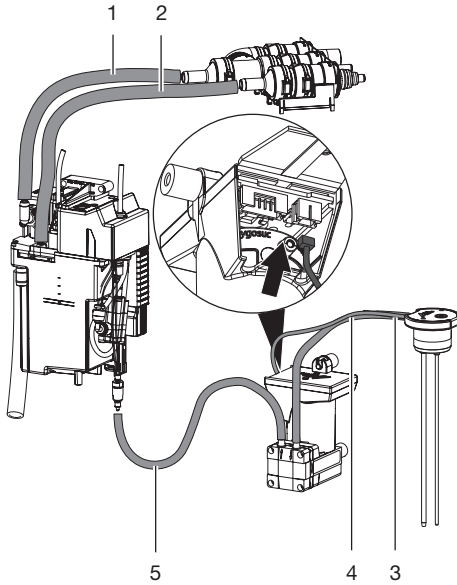


Fig. 5: Connection overview for modules with CDS 1

- 1 Connection of suction points for aspiration of the ready-to-use solution with disinfectant
- 2 Suction point connection for flush-cleaning
- 3 Cable, CDS 1 pump module to suction head
- 4 Suction head connection to the pump module
- 5 Pump module connection to the dosing module (disinfectant)



Properly route all hoses during installation. Look out for sharp edges, tight bends etc.


- › Connect the modules with hoses and secure them with the enclosed hose clamps.

7.4 Connecting the unit

- › Connect the compressed air supply.
- › Connect the water supply.
- › Connect the continuous rinsing system (optional).
- › Connect the power supply (24 V).

Connecting the device to the continuous rinsing system (optional)


- › Use adapter nozzle.

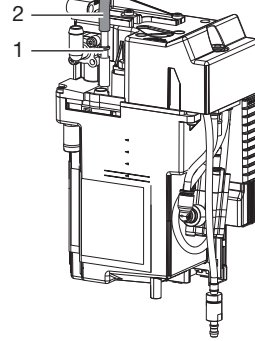

- 1 Adapter nozzle
- 2 Continuous rinsing connection

The amount of water for rinsing the hoses is reduced. Only as much water is aspirated as is re-supplied when the solenoid valve is opened.

- › When using the continuous rinsing function, observe the height difference between the continuous rinsing connection on the basic unit and the rinsing connection in the suction pipe.

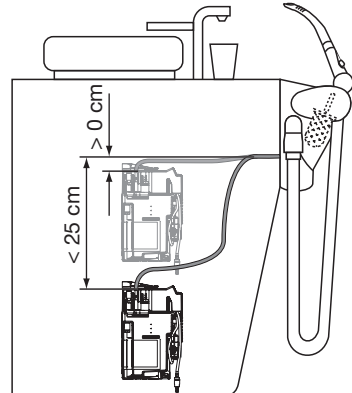

8 Commissioning

8.1 Function test

- › Check the hose connections for leak-tightness.
- › Perform flush-cleaning.
- › Press the button for dosing and supply of the ready-to-use solution with disinfectant.
- › Check that the overflow removes the maximum accumulating water volume.
- › Check that the accumulating water volume is removed with 5 bar water pressure "4.1 Dosing module".

8.2 Electrical safety checks

- › Carry out the electrical safety check according to the national law (e. g. in accordance with IEC 62353).
- › Document the results.

8.3 Combining devices safely

- › When connecting the unit to other devices, such as a PC system, comply with the requirements set out in section 16 of IEC 60601-1 (EN 60601-1).
- › When setting up the PC system in the vicinity of the patients:
Only connect components (e.g. computer, monitor, printer) that comply with the standard IEC 60601-1 (EN 60601-1).
- › When setting up the PC system outside of the vicinity of the patients:
Connect components (e.g. computer, monitor, printer) that comply at least with the standard IEC 60950-1 (EN 60950-1) at least.

8.4 Monitoring the unit via the network




Further information on Tyscor Pulse can be found in the software help and in the Tyscor Pulse manual, order number 0949100001.

The following requirements must be met in order to monitor the unit on the computer:

- Unit connected to the network
- Current monitoring software installed on the computer

Add device

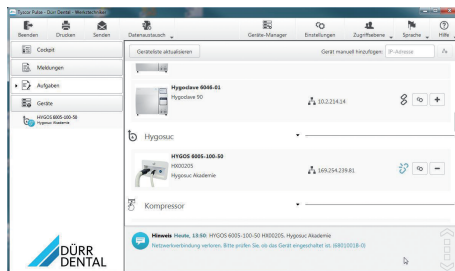

Connection icons:



The unit is present in the network and connected to the software.



The unit is present in the network but not connected to the software.




The network connection between the software and the unit has been interrupted, e.g. the device is switched off.

Requirements:

- ✓ Unit is switched on and connected to the network
- ✓ Administrator or service technician access level selected in the software
- › Working in the menu bar, click on **Device Manager**.
The list of units appears. A symbol displays the connection status to the software:
The new unit that is not yet connected, is displayed with the connection status .
- › Select the unit and click on .
The unit appears in the side bar.

Adding the device in the cockpit

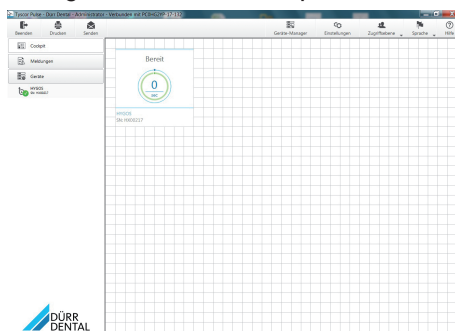

All devices that are connected to the software can be added to the cockpit. When the unit is

first connected to the software, the unit is automatically added to the cockpit.

Requirements:

- ✓ Administrator or Service Technician access level selected.
- › Click on the device in the device list with the left mouse button and keep the mouse button pressed.
- › With the mouse key pressed, drag the unit onto the cockpit.
- › Release the mouse key.
The block with the current characteristic data and the name of the device appear in the cockpit.
- › To change the position of the device block, click on the block and, with the mouse key pressed, drag it to the required location.

9 Operation

Requirements:

- The suction hoses are undamaged (no leaks).
- The supplied water is of drinking water quality (see "5.1 Dosing module").
- The unit is positioned horizontally.
- The connected pump is in operation.

9.1 Dosing and supply of a ready-to-use solution with disinfectant

Use daily after treatment is complete; with higher workloads, before the midday break and in the evening.

The disinfectant needs to react for one hour in order to achieve the desired level of disinfection. Do not use the suction unit on the treatment unit during this time.

i If the treatment unit is switched off during the reaction time then the operator must ensure that the required reaction time is still met.

Requirements:


- ✓ Disinfectant bottle full (see also "9.3 Changing the disinfectant bottle").

i Before pushing on the hoses, make sure that the cannula has been removed. In the event of contact between the suction connectors and the cannula they will need to be wipe-disinfected.


i Disinfection of the suction hoses is no substitute for cleaning and disinfection of the suction handpieces. Follow the cleaning instructions provided by the manufacturer of the suction handpieces.

- › Connect all suction hoses to the matching suction connections.

- › In the case of suction handpieces with a slider: open the slider.

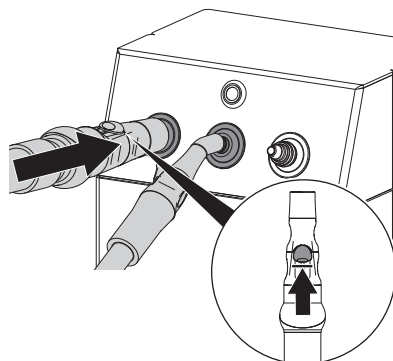

- › Press the button.

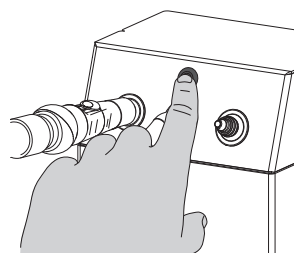

Cleaning begins. The button flashes green (rhythmically).

The suction hoses are rinsed 5 times with 200 ml water. Afterwards, 200 ml disinfectant solution is aspirated through the suction hoses 5 times. The reaction time of 1 hour begins. During the reaction time the button flashes green (non-rhythmically).

- › Remove the suction hoses and hang them in the hose manifold.

Otherwise the connected suction system will keep running in continuous operation.

i The reaction time can be interrupted at any time. To do this, press the button for approx. 2 seconds until the button lights up green (ready for operation).

- › Wait for the reaction time to elapse (one hour). The button then lights up green.



Use of the device is no substitute for regular disinfection of the suction connectors. The suction connectors need to be wipe-disinfected separately after each use.

**WARNING****Risk of cross contamination**

- › Before every treatment the suction handpieces must be disinfected.
- › Before every treatment the connecting pieces must be wipe-disinfected.

9.2 Flush-cleaning with water

Perform flush-cleaning on the suction hose used after each treatment.

- › Hold the suction hose on the blue flush-cleaning connection and hold it firmly in place here during the flush-cleaning.
Water is aspirated from the storage tank. If the storage tank is empty, the button flashes green (rhythmically).
- › Detach the suction hose from the flush-cleaning connection.

Result:

The storage tank refills with water. The button lights up green (ready for operation).

9.3 Changing the disinfectant bottle

Wear protective goggles.




Wear protective gloves.



During the disinfection cycle, the system will display if the disinfectant bottle needs to be changed. After replacing the bottle, restart the disinfection cycle.

- › Remove the empty disinfectant bottle from the treatment unit.
- › Remove the suction head from the empty disinfectant bottle.


- › Attach the suction head to the full disinfectant bottle.

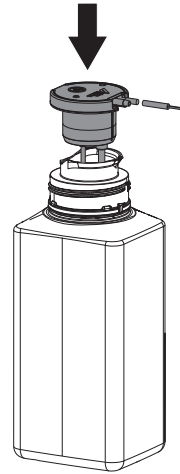

- › Insert the disinfectant bottle into the treatment unit.
- › Acknowledge the fault. To do this, press the button for 2 seconds.
- › Press the button to restart the disinfection.


9.4 Button with indicator light

Button lights up green

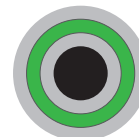


Device ready for operation

Button flashes green (rhythmically)

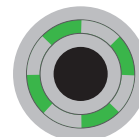


Device running

Button flashes green (non-rhythmically)

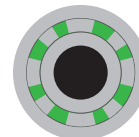

Reaction time running

Button flashes red (rhythmically)

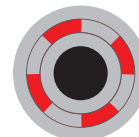

No disinfectant

Button lights
up red




An error has
occurred

9.5 Cleaning the storage tank

In the event of visible deposits on the internal wall of the tank, clean the storage tank with a cleaner that has been approved by Dürr Dental (e.g. MD555 cleaner, Dürr Dental).

Water amount / ml	Cleaning agent / ml
250	12.5

- Using a disposable syringe, add the cleaning agent to the water in the storage tank.
- After a reaction time of 30 min, aspirate the cleaning solution via the flush-cleaning connection.

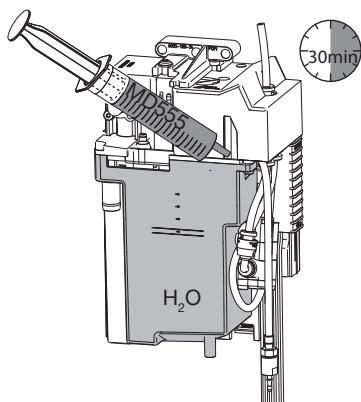

9.6 Monitoring the unit via the network



Further information on Tyscor Pulse can be found in the software help and in the Tyscor Pulse manual, order number 0949100001.

Monitoring operation

The device must have been added to the cockpit for the graphical device block to be shown in the cockpit.

Einwirkzeit



Shown in the unit block:

- Remaining run-time, Hygosuc
- Remaining run-time, reaction time

Querying messages



Normal operation



Fault
Operation of the device interrupted



Notice
Operation of the device restricted



Note
Important information about the device



Information



Establishing a connection to the device



Connection to the device interrupted

If a message occurs for an device, the symbol next to the device in the side bar changes. The message appears in the cockpit and in the device details.



If several messages occur, the symbol of the highest message level in each case is displayed.



As soon as a message concerning a device occurs, the symbol in the task bar (or Mac OS menu bar) also changes to the relevant message symbol. If required by the message an acoustic signal also sounds.

- › To query the message details, switch to the cockpit or to the device.


Creating a report

You can print out a current report  or sent it via e-mail .

The report contains all messages and a screenshot of the view that is displayed when the report is created.

9.7 Performing a visual check of the mixing ratio

- › Check the fill level of the Orotol. The dispensing container must be filled to at least the marker.
- › Check the fill level of the water. Do not fill the water container past the top marker.
Fill the water container so that the water level is between the two markers.

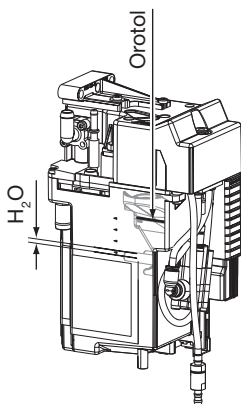

EN 10 Maintenance



Only trained specialists or personnel trained by Dürr Dental may service the device.



Prior to working on the unit or in case of danger, disconnect it from the mains.

Inspection interval	Inspection work
Annually	<ul style="list-style-type: none"> › Check the water connection for correct and secure seating. › Check the Orotol connection for correct and secure seating. › Check the mixing ratio, see "9.7 Performing a visual check of the mixing ratio". › Remove and, if necessary, clean the coarse filter for the water connection.
Maintenance interval	Maintenance work
Every six months	<ul style="list-style-type: none"> › Clean the inner wall of the tank with a cleaner that has been approved by Dürr Dental (e.g. MD 555 cleaner, Dürr Dental), see "9.5 Cleaning the storage tank".

? Troubleshooting



Wear protective goggles.



Wear protective gloves.

11 Tips for operators



Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.

Error	Possible cause	Remedy
Button flashes red	No disinfectant	<ul style="list-style-type: none"> › Check the disinfectant supply. › If necessary swap the tank. Then press the button for 2 seconds and restart the disinfection cycle.
Button lights up red	Suction hose has been pulled off the connection	› Reconnect the suction hose to the connection. Press the button for two seconds. Restart the disinfection.
	Insufficient suction power	<ul style="list-style-type: none"> › Check the compressed air supply $3 \text{ bar} \leq p \leq 5 \text{ bar}$? › Is the suction unit running? › Inform a Service Technician.
	Storage tank does not fill with water	<ul style="list-style-type: none"> › Check the water supply $3 \text{ bar} \leq p \leq 5 \text{ bar}$. › Check the water connection. › Inform a Service Technician.
	Leak at the suction hoses	› Inform a Service Technician.
	Dosing module defective	› Inform a Service Technician.
	Suction hose was held onto the suction connectors for flush-cleaning for longer than 60 seconds	<ul style="list-style-type: none"> › Remove the suction hose from the connection. › Press the button for two seconds.

12 Tips for service technicians



The following information about troubleshooting is intended solely for service technicians. Repairs must only be carried out by service technicians.

Error	Possible cause	Remedy
Button flashes red	No disinfectant	<ul style="list-style-type: none"> › Fill up with disinfectant. › Check the supply hoses for leaks/permeability. › Check the compressed air supply: <ul style="list-style-type: none"> › Check the interface. › Is the CDS 60 or CDS 1 working? › Pressure $3 \text{ bar} \leq p \leq 4 \text{ bar}$?
	Water mixture is not aspirated	<ul style="list-style-type: none"> › Check the compressed air supply $3 \text{ bar} \leq p \leq 5 \text{ bar}$? › Check the interface. › Does the green LED light up on the PCB? › Is the suction unit running?
Button lights up red	Storage tank does not fill with water	<ul style="list-style-type: none"> › Check the interface. › Is there a leak in the water line or is the water line blocked? › Is the coarse water filter blocked? › Check the "water fill level" calibration.
	Dosing module defective	<ul style="list-style-type: none"> › Replace the dosing module.

13 Error messages in Tyscor Pulse

Error	Possible cause	Remedy
Disinfectant missing. Replace the disinfectant container and confirm the message. Check the disinfectant supply lines if necessary.	No disinfectant	<ul style="list-style-type: none"> ➤ Fill up with disinfectant. ➤ Check the supply hoses for leaks/permeability. ➤ Check the compressed air supply: <ul style="list-style-type: none"> ➤ Check the interface. ➤ Is the CDS 60 or CDS 1 working? ➤ Pressure $3 \text{ bar} \leq p \leq 4 \text{ bar}$?
Disinfection failed. Suction interrupted or insufficient. Repeat disinfection. Check the suction performance on the suction hoses and the compressed air supply. Inform a Service Technician if necessary.	Water mixture is not aspirated	<ul style="list-style-type: none"> ➤ Check the compressed air supply $3 \text{ bar} \leq p \leq 5 \text{ bar}$? ➤ Check the interface. ➤ Does the green LED light up on the PCB? ➤ Is the suction unit running?
No water supply	Storage tank does not fill with water	<ul style="list-style-type: none"> ➤ Check the interface. ➤ Does the green LED light up on the PCB? ➤ Is there a leak in the water line or is the water line blocked? ➤ Is the coarse water filter blocked? ➤ Check the "water fill level" calibration.
Water fill level too high. Inform a Service Technician.	Water fill level too high	<ul style="list-style-type: none"> ➤ Stop the water supply, check the operation of the water valve.
Sensor malfunction	Dosing module defective	<ul style="list-style-type: none"> ➤ Replace the dosing module.



Hersteller / Manufacturer:

DÜRR DENTAL SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
Fon: +49 7142 705-0
www.duerrdental.com
info@duerrdental.com

